

Respireo SOFT Nasal



Enjoy every moment



The success key helping you to support patient acceptance in their treatment .
The Respireo SOFT series will make your life and those of your patients simpler through a unique and optimal user experience.

Performant

- Few references for optimizing the inventory management as well as the preparation of the patient visits
- Quick set-ups and fitting to help you to get the right fit since the first time

Comfortable

- Cushion with SOFT technology fitting the contours of the nose to reduce pressure on the nasal bridge, which is a highly sensitive area
- The hooks position close to the face to enhance stability and air tightness with gentle settings of the headgear
- Open field vision for better independence

Easy to use

- Easy to install, adjust and clean.
- Preservation of the headgear's settings (pull the headgear off through the notches on the forehead support and the hooks on the lateral arms)



The Univeo headgear

is designed to self-locate high on the head in order to distribute the support and offer a greater stability

Light and durable frame

Limit disturbances to the seal during the night to obtain airtightness while ensuring a clear line of sight.

Respireo SOFT cushion

naturally fits the nasal bridge for a high level of comfort and seal night after night

Ergonomic hooks

Making easy for patients to remove the mask and lock it in the same fit

Flexible lateral arms

Auto-adjust to facial width for a perfect stability without requiring any tension of the headgear




Diffused and quiet venting

directs air away from the patient and his bed partner

360° rotating elbow


With a quick release swivel for freedom of movements

Instructions of use

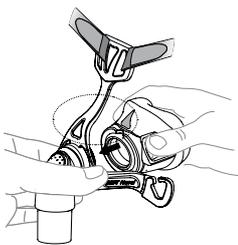


- 1 Align the two triangles and clipse the cushion on the frame.


Tips: The two lines under the cushion and frame should also be lined up.

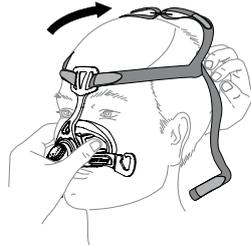

- 2 Position the mask on the face (upper part of the cushion at 2/3 of the nose) while pulling the headgear over the head

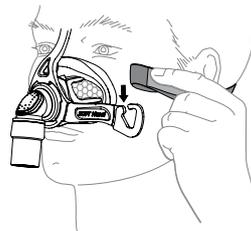


- 3 Loop the lower straps of the headgear into the hooks

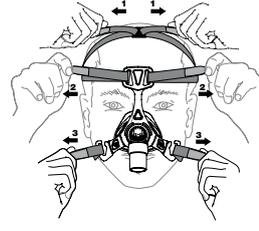

- 4 Adjust the settings of the headgear from top to down (1, 2 then 3).

Tips: Ensure to not tight too much the headgear

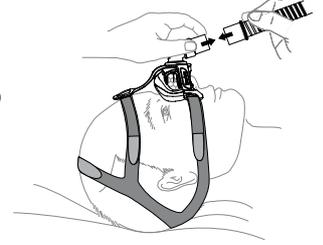

- 5 Make final adjustment while lying down. Softly rise the cushion until it naturally sits on the bridge of the nose. If needed, gently release/tighten the headgear settings.

For complete user instruction refer to the User Guide or the video on YouTube : Respireo SOFT Nasal

Codes

KM227500	Respireo SOFT Nasal size M	(manufacturer reference 472360*)
KM227700	Univeo headgear	(manufacturer reference 472471*)
KM227800	Respireo SOFT Nasal Cushion size M	(manufacturer reference 472470)

Respireo SOFT
Nasal

Please read the user manual
(*) Medical device class IIa CE 0051. Manufactured by Air Liquide Medical Systems Spa.
Distributed by Air Liquide Medical Systems SA.

AIR LIQUIDE
Healthcare